



MODEL 900R BARIATRIC RECLINER



Warning, Watch for Pinch Points

Keep fingers, arms, legs and objects clear when any mechanism is in motion. Make sure all persons and objects are clear when raising or lowering the back.

The Bariatric Recliner Chair is equipped with 2 brake casters (back of chair) and 2 steer casters (front of chair). When entering or exiting the chair make sure the brakes are engaged. The steer caster will lock for pushing the chair in a straight line.

To operate the recliner pull up on the release handle and push down on the back to the desired position then release the handle. As the back comes down the foot rest will go up.

Caution: Before using any cleaning product, test it on the underside of chair. Never use any volatile or flammable fluid (thinner, gasoline, etc.) that could cause discoloration or deterioration.

Cleaning

Use mild soap and water or a nonabrasive hospital-grade detergent disinfectant to clean the chair. Wet a cloth with the cleaning product and wring dry. Then gently wipe off the chair.

Periodic Inspection and Maintenance

The amount of maintenance required on the chair will be dictated by its use.

Depending on usage, inspect the chair after one to two weeks of usage and then at a minimum of every six months.

Before two weeks and then every six months minimum thereafter

Check all the bolts to be sure that none have worked loose.

Check that pins are securely fastened.

Check all welds.

ConvaQuip Ind., Inc.
P.O. Box 3417
Abilene, TX 79604
Phone 800-637-8436