USER GUIDE 1-6

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COMODITÀ

*Excellence in design and safety*

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ROLLING WALKER - indoor

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model number: COM 800

COMODITÀ is a company concerned with your comfort, safety and mobility. Our products will simplify your movements, offering support and safety for your daily activities, helping you do what you want, when you want it.

COMODITÀ Avanti is an elegant, modern, patented, light weight rolling walker that can be operated with one or two hands.

The brake bar on COMODITÀ Avanti is placed under the entire soft rubbery handle, allowing you to brake both rear wheels simultaneously with only one hand.

By pushing the brake bar down you can lock the brake when you want to park your walker.

In addition to the exclusive COMODITÀ Pisa removable cup holder, our Avanti model has a removable plastic tray that can be used to transport small items.

Under the tray you have a very roomy, removable, washable, nylon bag for your personal items.

The COMODITÀ Avanti rolling walker has four 8" wheels, making it very easy to maneuver.

The COMODITÀ Avanti can also be easily folded for storage or transportation.

The stylish and contemporary design of COMODITÀ Avanti, with our signature "S" shaped back wheel support provides extra space between the rear wheels helping to avoid trips and falls, making it perfect walking aid, even on narrow and small spaces.

**Please do not hesitate to contact us through our website: [www.comoditauusa.com](http://www.comoditauusa.com), if you have any doubts, questions or problems, as we are at the disposal of our customers!**

## Other Features:

- Unique one or two hand operation
- Brake bar activates both rear wheel brakes
- Handle finished with soft EVA foam for added comfort
- Adjustable handle height
- Easy to operate in narrow and small spaces
- Convenient removable cup holder
- Removable plastic tray to transport small items
- Removable, washable nylon bag to transport personal items
- Our signature "S" shaped back wheel support provides extra space between rear wheels to avoid tripping and falling
- Light weight (15 lb) and easily foldable for transportation
- Hand brake locking system for parking
- Large 8" wheels help overcome small obstacles

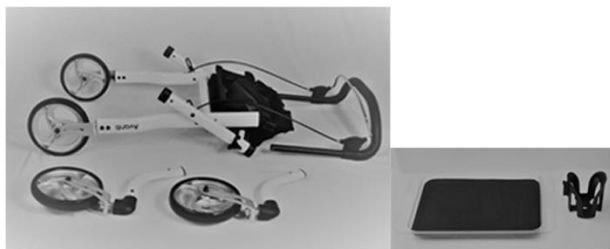
## Product Dimensions and Specifications:

- Unit weight: 15 lb.
- Weight capacity: 300 lb.
- Weight capacity for tray and nylon bag: 20 lb
- Recommended user height: 4'8" to 6'2"
- Overall height: 33.0" / 37.0"
- Overall length: 27½"
- Overall width: 23.0"
- Folded width: 18.0"
- Wheel diameter: 8.0"
- Height of tray from floor: 26 ½"
- Size of nylon bag: 15.0" X 10.0" X 6 ½"
- Size of tray: 15 ½" X 10.0"


## Assembly Instructions:

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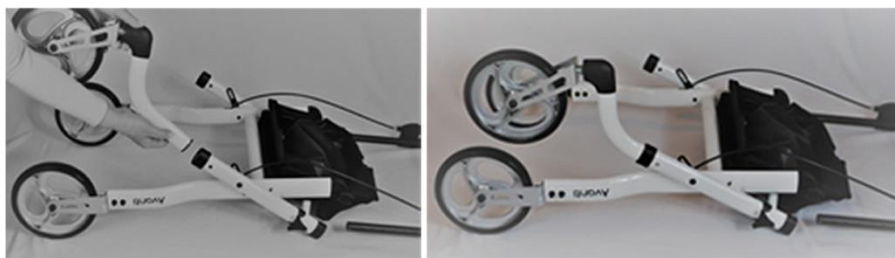1. Insert the front wheels into the frame by pushing the frame of the front wheel into the main frame of the walker. The metal buttons will lock at the designated position.

2. Open the main frame of the walker by pushing apart the front and back wheels supports.

## Assembly Instructions:

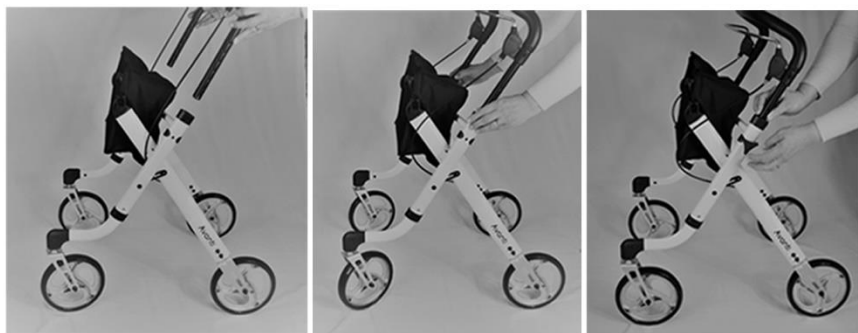
3. Remove the plastic knobs that are located on both sides of the main frame of the walker.

4. Insert the handle bar into the top side of the frame. Make sure you insert both sides simultaneously; otherwise the handle bar will not slide into the frame smoothly.

Adjust the height of the handle bar to a point you feel comfortable operating the walker.

Insert the metal screw into the frame to lock the handle at the desired height. Make sure the height of the handle is the same on both sides by matching the numbers on the handle bar.

## Assembly Instructions:

5. Lock the metal support of the plastic tray into the main frame.

6. Insert the plastic tray into the metal support.

7. Insert the cup holder into the cup holder support.

**It is that simple!**  
**No tools required!**  
**Now you are ready to go!**

## Operating Instructions:

### To lock and unlock brakes

1. Push the brake bar down until you hear a click.
2. Pull the brake bar up to unlock the brakes.

### To transport the Walker

1. Remove the plastic tray

2. Unlock the tray support from the frame

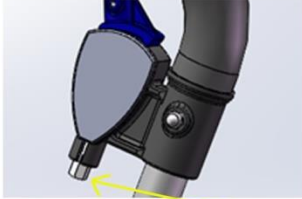
3. Move the front wheels toward the back wheels.  
Your Comodità Avanti was designed to stand alone when closed for transportation.

## To Adjust the Brakes:

To adjust the sensitivity of the break:

Locking nut

1. Turn the locking nut clockwise to increase brake sensitivity.
2. Turn the locking nut counter-clockwise to reduce brake sensitivity.

## Safety Instructions: Warning! Failure to follow safety instructions may result in serious injury.

- **DO NOT** exceed maximum weight capacity.
- **INSPECT BRAKES** before each use.
- **ALL WHEELS MUST** be in contact with the floor or ground at all times.
- **DO NOT** use in escalators, stairs, moving sidewalks, or uneven surfaces.
- **RECOMMENDED USER'S HEIGHT:** from 4'8" to 6'2".

**Warranty:** Your Comodità Rolling Walker is warranted to be free of defects in material and workmanship for a period of one year from the date of purchase for the original owner. This warranty does not cover device failure due to user negligence, misuse of the product, failure to adhere to instructions, or normal wear and tear. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not include any labor or shipping charges. Please contact our customer service at: [info@comoditaua.com](mailto:info@comoditaua.com). We will provide a return authorization number.

You will be required to include a copy of your original receipt.

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COMODITÀ LLC USA