USER GUIDE 1-6

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COMODITÀ

Excellence in design and safety

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ROLLING WALKER WITH
ORTHOPEDIC SEAT

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The modern design and sturdy construction of the Comodità Prima Special Rolling Walker make it one of the safest walkers on the market. The exclusive 16" wide orthopedic seat makes it ultra comfortable, ideal for users that enjoy resting between walks. Built to support weights of up to 400 lbs (182 kgs), it is tested and approved under the most demanding standards for rolling walkers. Our signature "S" shaped back wheel support provides extra room for the feet to move, helping to avoid tripping and falling. It also allows the weight to be placed right above the wheels, and not on the side of the wheels like most models on the market. Our rolling walker is built with the best materials available to ensure the highest safety you can find.

COMODITÀ Prima Special wheels are placed directly under the frame, allowing the full weight of the user to be placed directly on the wheels.

On other models the wheels are attached to the frame on the side, which is less supportive and less stable.

Other Features:

- Exclusive ultra comfortable 16" wide orthopedic seat with coccyx relief contour.
- Heavy-duty 400 lb weight capacity.
- Reinforced frame for maximum safety.
- Extra large 8" wheels.
- Easy two-step assembly - no tools required.
- Easily adjustable handle height.
- Hand-brake locks for safety while seated.
- Exclusive COMODITÀ Pisa cup holder included
- Our signature "S" shaped back wheel support provides extra wide space between rear wheels to avoid tripping and falling.
- Easily folds for storage and transport.
- Extra wide ergonomic backrest.
- Convenient under seat storage bag.

Product Dimensions and Specifications:

- Weight Capacity: 400 lb
- Recommended User Height : 5'4" to 6'4"
- Overall Height: 35.0" to 39.0"
- Overall Length: 28.0"
- Overall Width : 25 ½"
- Seat Depth: 14 ¼ "
- Seat Width: 16.0"
- Seat Height From Floor: 22 ½"
- Folded Width: 10.0"
- Wheel Diameter: 8.0"
- Unit Weight: 19 ½ lb



Assembly Instructions:

Assembly your COMODITÀ Prima Special Rolling Walker is extremely simple.

1. Carefully remove the walker from the shipping carton.
2. Unfold the walker by pushing down the folding brackets on both sides of the rolling walker until they are fully extended.
3. Insert handle bars into the frame of the walker, keeping the brake cables outside the frame.
4. Adjust the height of the handles by standing upright behind the walker with your shoulders relaxed and arms hanging at your side. The handgrips should be at a height even with your wrists.

It's that simple. No tools, no loose parts.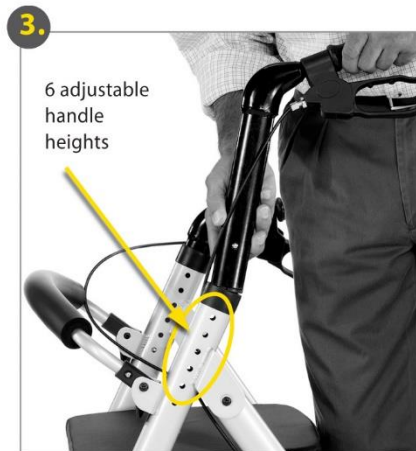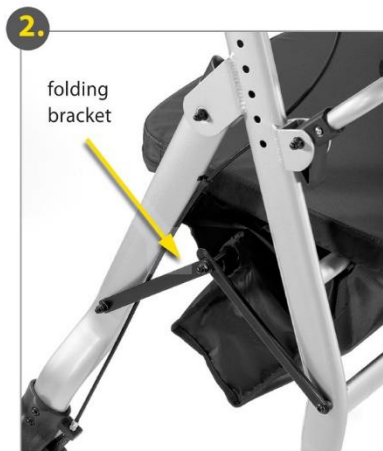

Operating Instructions:

To walk

- Position yourself behind the walker and hold both handles firmly.
- Roll your walker slowly forward so that the rear wheels are a few inches ahead of your body.
- While maintaining a steady and safe roll forward, place your foot in line with the rear wheels.
- Repeat the movement with your other foot.
- To slow down, pull the brake handles up.

To lock and unlock brakes

- To lock the brakes, push down the brake levers until the brakes click into locked position, as shown below.
Do not push it any further than this point.
- To unlock the brakes and resume walking, pull up the brake levers until you hear a “click”.

To use as a seat


- Make sure your walker is on a flat surface, not on an incline.
- **Lock** the brakes before sitting.
- Make sure your feet rest flat on the ground.

Care & Maintenance

Periodic brake adjustment

To adjust the sensitivity of the brake:

- Turn locking nut clockwise to reduce brake sensitivity.
- Turn locking nut counter-clockwise to increase sensitivity.

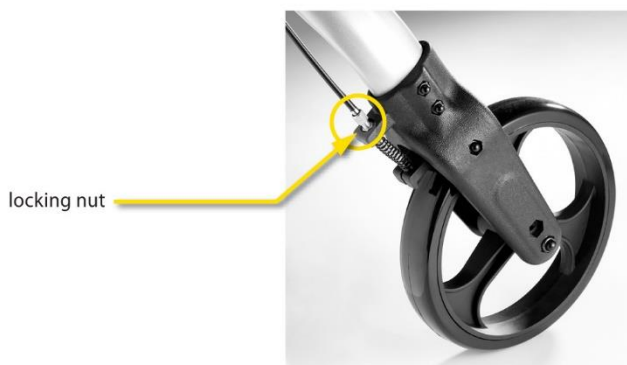Warning!

- Over adjustment of the brake might prevent the wheels from moving freely.
- Under adjustment of the brake may cause the brake pad not to engage at all.
- ***Do not use the walker if brakes are not working properly.***

Safety Instructions:

Warning! Failure to follow safety instructions may result in serious injury.

- DO NOT use this product as a wheelchair.
- DO NOT transport someone while sitting on this walker.
- DO NOT exceed maximum weight capacity.
- BRAKES MUST be in locked position before and sitting on the seat.
- DO NOT self-propel the walker while seated.
- DO NOT sit on the walker on an incline.
- INSPECT BRAKES before each use.
- ALL WHEELS MUST be in contact with floor or ground at all times.
- BACKREST IS NOT intended to support your full weight.
- DO NOT use in escalators, stairs, moving sidewalks or uneven surfaces.
- BOTH FEET should rest flat on the ground when sitting.
- RECOMMEND USER'S HEIGHT from 5'4" to 6'4".
- Your COMODITÀ rolling walker should be revised periodically to ensure that the brakes are working perfectly and that all nuts and bolts are tight.

Warranty:

Your COMODITÀ Prima Special Rolling Walker is warranted to be free of defects in material and workmanship for a period of one year from the date of purchase from the original owner. This warranty does not cover device failure due to user negligence, misuse of the product, failure to adhere to instructions, or normal wear and tear. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not include any labor or shipping charges. Please contact our customer service at info@comoditauusa.com. We will provide a return authorization number. You will be required to include a copy of your original receipt.

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COMODITÀ LLC USA