



CHROME PLATED STEEL GRAB BAR W/ KNURLED GRIP



USER MANUAL - CHROME PLATED STEEL GRAB BARS

GENERAL EXPLANATION



⚠ Professional installation required



Grab Bar (1)



Screws (6)

- Knurled finish for a better grip
- 1 ½" distance from wall
- Mounting hardware included
- Supports up to 250 lbs
- Meets ADA & CSA Standards for Accessible Design

INSTALLATION INSTRUCTIONS

- ⚠ Carefully read through all instructions before using and installing Grab Bars. This product requires professional installation.
- ⚠ Check that there is no plumbing or electrical wiring in the installation area. Screws MUST be mounted into WOOD STUDS.

- 1 Choose the Grab Bar location most convenient to user. May be placed vertically, horizontally or diagonally.
- 2 Find wall studs and place grab bar on wall in line with studs and mark where screws will go in. After choosing a (vertical, horizontal or diagonal) position, drill holes for screws, ensuring the holes are in the stud. Use only wood screws for attaching to wood studs.

WARNING: Your professional contractor should use at least two wood screws in each flange to ensure that the grab bar is securely attached and strong. If it is not securely attached, using the grab bar may cause serious personal injury.

- 3 **Tools required for this step:**
Electric drill and masonry bit

After holes are drilled, place grab bar in position and attached with screws. You can also use toggle bolts for this step. If using toggle bolts (not included) ensure bolt is attached before putting it into hole. For best results, caulk the edges of the flange, to keep water and moisture out.

Using a screwdriver, tighten all screws and bolts. Periodocially, check screws are secure and tight. Do NOT use grab bar if it is not secure to the wall. This can result in serious personal injury.

OPERATING INSTRUCTIONS

 Users with limited physical capabilities must be assisted or supervised when using a Grab Bar.

- 1 Grasp the Grab Bar firmly when exiting or entering bathtub or when sitting or rising from commode.
- 2 Use EXTREME caution when standing on wet surfaces.
- 3 Grab Bar is NOT designed to support the total weight of an individual. Use the Grab Bar for assistance only.

SAFETY PRECAUTIONS

If you are not able to perform installation as described, please contact a licensed professional. Lifestyle Mobility Aids, Inc. assumes no responsibility for any damage or injury caused by improper installation or use of this product. If you do not fully understand the operation instructions for this product, please contact a healthcare professional for guidance.

WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT substitute parts.

MAINTENANCE

 Inspect Grab Bar weekly and tighten as needed to ensure securely installed. DO NOT use Grab Bar if loose.

› Clean with soap and water or a mild detergent and rinse with water. Ensure Grab Bar is rinsed well and dry before use.

NOTE: Never use abrasive cleaners or scouring pads on any part of the Grab Bar.

WARRANTY

Limited Lifetime Warranty is provided against any factory defects in material or workmanship of the product frame.

One (1) year warranty against defective materials or workmanship on all other components, assemblies and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation or unauthorized modifications.

This warranty does not cover non-durable components such as rubber accessories which are subject to normal wear and need periodic replacement and/or adjustment.

It is recommended but not required that the product be returned to the Lifestyle Mobility Aids dealer through whom it was originally purchased for warranty service. However all Lifestyle Mobility Aids dealers are qualified to assist you in obtaining warranty service. If the product is to be returned to Lifestyle Mobility Aids, prior authorization is required and Lifestyle Mobility Aids will determine if the product needs to be repaired or replaced.

If you have a question about your Lifestyle Mobility Aids product or this warranty please email us at contactus@lifestylemobilityaids.com.

Rhythm Healthcare, LLC
Lifestyle Mobility Aids
3200 Tyrone Blvd N
St. Petersburg, FL 33710

contactus@rhythmhc.com
Website: www.rhythmhc.com
(877) 843-6464

V2: 11/30/21