

# Standard Seatback Bag

B1111

DIESTCO Standard Seatback Bag mounts to the seatback of your scooter or wheelchair. Spacious, single compartment bag. Ideal for everyday use. Made from high quality, water-resistant polyester with reflective safety tape, visible up to 300 yards.

Fits seatbacks 7" - 24" tall.

## Mounting Instructions:

- 1) Remove the strap from the metal loop.
- 2) Pass the strap through the opening between the seat and the seatback.  
Be sure to pull the strap through completely and that it is not twisted.
- 3) Re-insert the strap through the metal loop and pull through.
- 4) Pull strap down tightly and fasten the loop end to the hook portion of the strap.
- 5) Excess strap can be cut once you have your Standard Seatback Bag mounted.

**If you have any issues mounting the bag, please call us at (800)795-2392 for assistance.**



DIESTCO Mfg. Corp.  
P.O. Box 6504  
Chico, CA 95927  
(800) 795-2392  
[www.diestco.com](http://www.diestco.com)

We strongly encourage you to call us at (800)795-2392 if you have any problems, questions, or comments. We are always developing new products for wheelchairs, scooters, crutch, and walker users...so let us know if there's something special you need.