

Archimedes

Battery Charging

- Disconnect the hand control from the actuator.
- Connect the hand control to the battery charger (fig.12).
- Plug the battery charger into a mains power supply outside of the bathroom and switch on. The amber light on the charger (**A** fig.12) will light up to indicate that the charger is properly connected and charging.

When the battery needs recharging, the hand control should be left on charge over night. The hand control can be left on charge continuously without damaging the battery or charger. **We recommend that the hand control is left continuously on charge when the Archimedes is not in use, as a matter of routine.**

