

LIFETIME LIMITED WARRANTY

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.



3 Wheel Aluminum Rollator 300lbs Weight Capacity



MH3RLBE - Blue



844604015882

MH3RLRE - Red



844604015899

MOBB Health Care USA LLP
59 SW 12th Ave #101 Dania
Beach, FL, 33004
Tel: (754) 210-3130
Fax: (754) 263-3978
www.mobbhccusa.com

MOBB Health Care Ltd.
116A Viceroy Rd. Concord,
ON L4K 2M1 Canada
Toll: 1-866-238-7190
Tel: 905-669-0348
Fax: 905-669-2796
www.mobbhcc.com

INSTRUCTIONS INSIDE

Assembly Instructions

Please read these instructions before assembly or use of Rollator. This Rollator must always be assembled Authorized and fitted for the user by a qualified dealer. Before using your new Rollator please ensure that your weight is within the weight assembly capacity of this unit. This Rollator will require very little assembly and assembly can be completed without tools.

1. To open your rollator, pull both side frames outward. Push the folding hinge with the black handle until straight to secure the unit in the proper position.

2. Place a handle bar in each handle bar receptacle and adjust to the desired height. The brake cable should remain on the same side as the corresponding wheel. Do not cross cables. Insert locking knobs into adjustment holes and tighten to secure handle in place, making sure both handles are at equal height.

3. Slide wheels back until push pins pop through the adjustment hole located on the bottom section of the wheel frame. Lock wheels in place by inserting the locking knobs into the top section of the wheel frame and tightening until secure (figures 1 & 2)



4. Test and adjust the loop lock mechanism tension to ensure proper and secure locking before use.

The way of activating the parking loop lock is by pushing the handle down. This way is convenient for people who cannot easily pinch the handle, because of certain disabilities of their hands. Releasing the parking loop lock can be done by moving the loop lock handle up. For its operation the provided with a parking wider loop section lock, at the the end.



NORMAL LOOP LOCK

Normal loop lock is activated by pulling the handle UP, similar to operating the brakes of a will bicycle. Once the handle is released, the lock will release as well.



PARKING LOOP LOCK

Release the parking loop lock by pulling the handle UP. The handle will return to its basic position; the parking loop lock is OFF.

Instructions to Adjust the Hand Break System Important Safety Notice

The hand brake system fitted on this rollator complies with the manufacturer's quality standards. The hand brake system has been preset (adjusted) to the best performance settings. To better suit you personally, please follow the instructions set out below. You may need to adjust the hand brake systems from time to time as they will wear out with normal use.

