

Lifetime Limited Warranty

If you have a question about your MOBB Healthcare device or this warranty, please contact an authorized MOBB Healthcare dealer.

MOBB Healthcare warrants to the original consumer purchaser for as long as they own their MOBB product that it will be free from defects in material and workmanship under normal use. This warranty is only applicable to products used in residential applications within the United States and Canada and is not transferable to subsequent owners. Should defects in material or workmanship exist during the applicable warranty period in any MOBB Healthcare Product, MOBB Healthcare will elect at its discretion to either repair or replace the defective material. This warranty covers only parts and materials of the products supplied by MOBB Healthcare. Not covered under this warranty are wear and tear items or abuse of the product in a manner that was not intended for the product to be used. This warranty shall become void if the product is in any way modified, improperly adjusted or abused.



Bed Assist Rail

Weight Capacity: 300lbs



INSTRUCTIONS INSIDE

MHHBR



844604078146




MOBB Healthcare Ltd.

116A Viceroy Rd. Unit 5-6
Concord, Ontario, Canada
L4K 2M1
www.mobbhbc.com

ASSEMBLY and INSTALLATION

1. Slide the straight bars (A) over the long end of the "L" shaped bars(B) until the button lock.
2. Slide the assembled bars(buttons to the outside of assembly) into hand bar (C).
3. Slide the end bar (E) into the bar (A) until the buttons lock the bars together.
4. Ensure all clocking buttons are protruding through their respective holes.
5. Slide the assembled unit between the mattress and box spring at the point along the bed best suited for the user.
6. Adjust hand bar height using locking buttons below handgrip foam. **Warning:** The distance between the hand bar & mattress should not exceed 4.5"(120 cm).
7. Position safety rail at desired position. The space between the vertical handle bars(F) and other bedside device or bed board must be less than 2.25"(6 cm), or larger than 9.25"(23.5 cm).

Figure 1

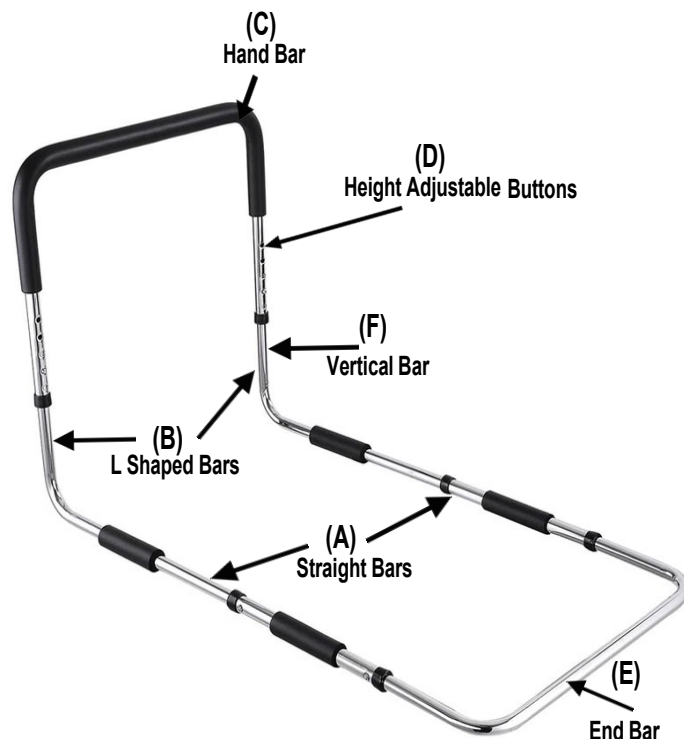



STRAP INSTRUCTIONS

1. Secure the SAFETY STRAP to the bed frame, as shown in Figure 2. Unbuckle strap and loop strap around bed frame and re-buckle. Tighten strap to secure Handle Assembly against the side of the mattress.
2. DO NOT INSTALL PRODUCT WITHOUT SECURING THE SAFETY STRAP TO THE BED FRAME.
3. 1 strap included.

WARNING

- This Bed Rail can be installed on most traditional bed types with mattresses 12" to 16" thick. This Bed Rail is not intended for use on adjustable beds or with extra soft mattresses.
- This Bed Rail is not designed or intended to keep a person from falling or rolling out of bed. It is designed to assist the user in entering and leaving the bed by providing a safe handhold.
- Do not use this Bed Rail without the properly connecting the supplied safety strap.
- **Use without the safety strap can allow the bed rail to slip out of position, creating a dangerous gap between the handle and the mattress side posing a risk of entrapment, strangulation and death.**

Figure 2

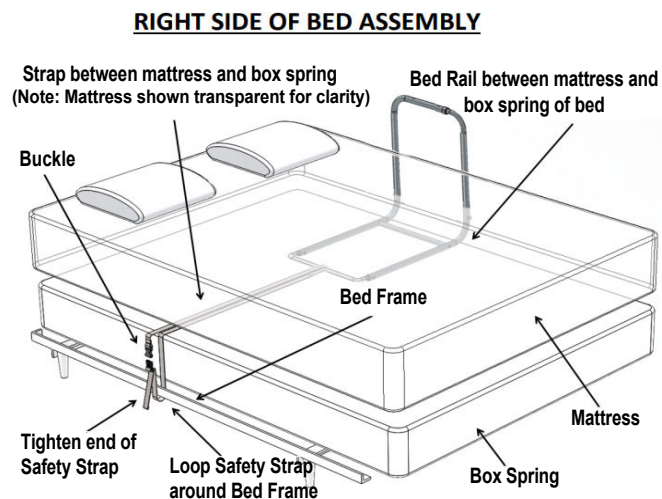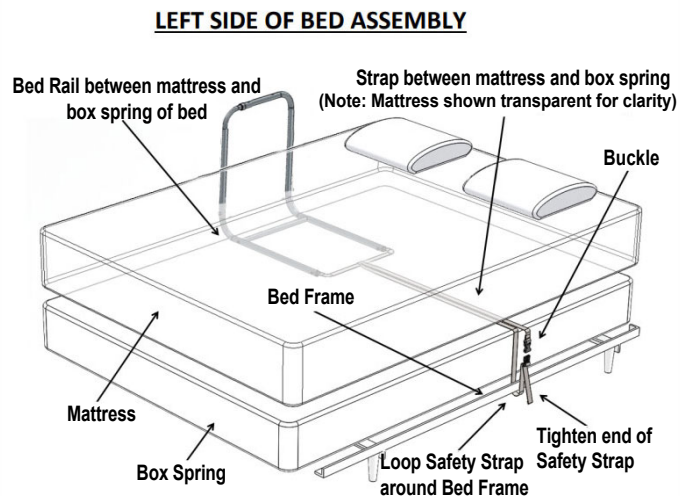

Bed Rail Safety Strap Buckle Instructions:

Align the buckle as shown in figure 1

Figure 1



Insert the buckle as shown in figure 2

Figure 2



Insert screw and tighten with screw driver as in figure 3

Figure 3

