


Side skirt with 90 degree angle pool floor and cut corner option



Step 1. Please read the instructions in full prior to construction. It really helps. First determine which type of pool floor you have. If you have a curved pool floor build as seen in Photo 2.

Note: You will need to have a rubber mallet to tap/hammer the pipe into the fittings.

Step 2. Lay out the pipe as shown in Photo 3 for the 90 degree angle pool floor. For the curved pool floor locate the 2 angled fittings and short pipe for the lower corner and assemble as seen in photo 2.

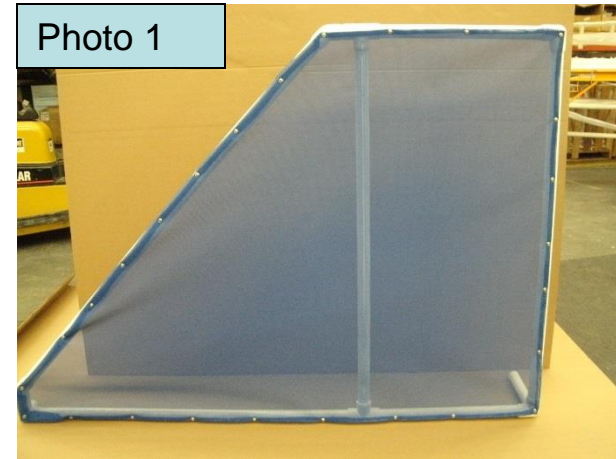

Note: Side screens may differ for custom systems.

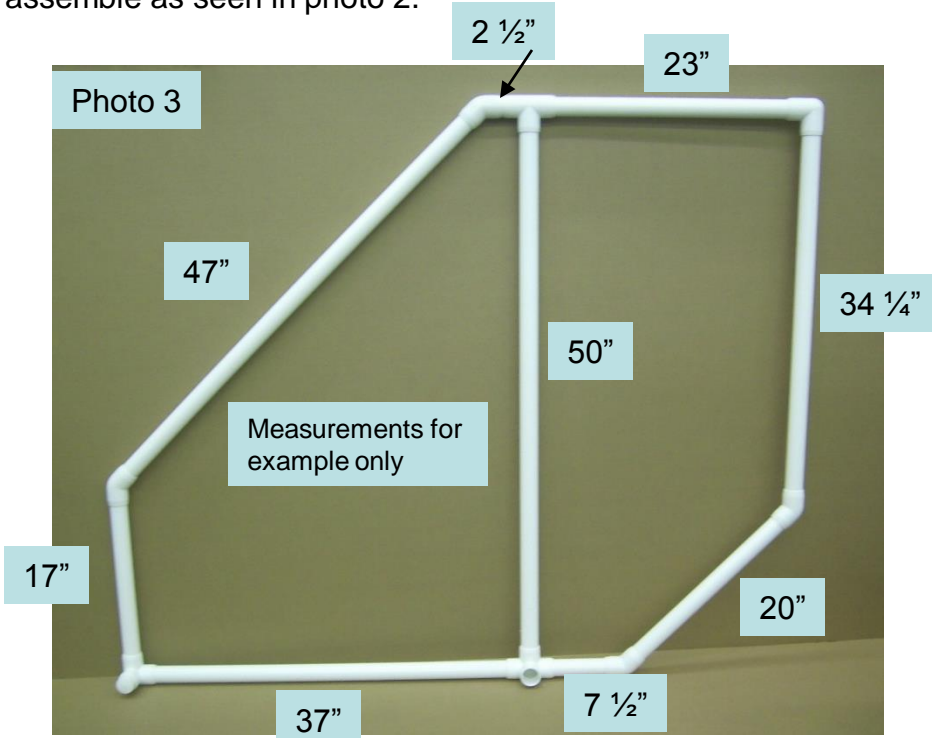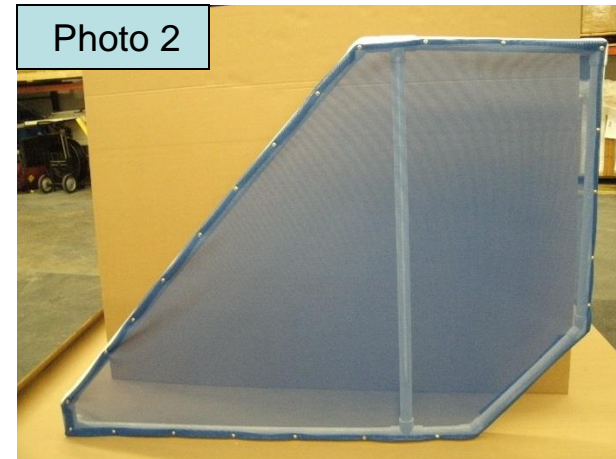

NOTE: The direction of the lower fittings will determine if the screen gets placed on the Left or Right side of the step system. Please verify which side of the system the screen is being placed prior to construction.

Step 3. Use the 2 1/2" pipe in the lower left fitting as shown in Photo 5. You will need to use a rubber mallet to hammer the pipe in. Use the other 2 1/2" pipe at the top center angled fitting as shown in Photo 4. Be sure to hammer the pipe into the fittings with a rubber mallet.

Use a rubber mallet to hammer the pipe into the fittings.

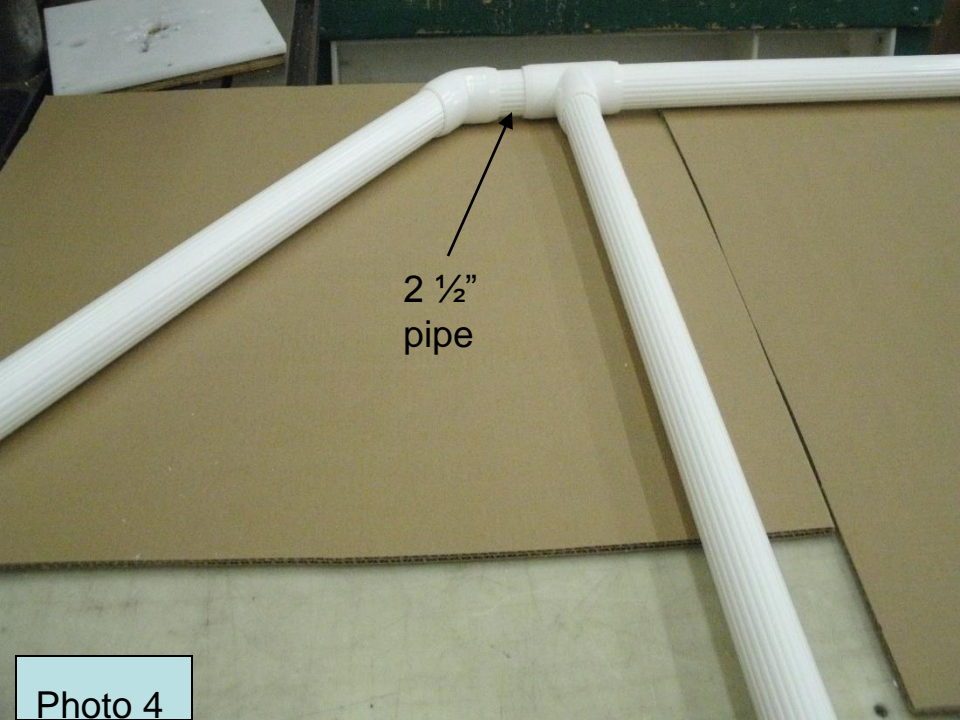


Photo 4

Photo 5




Use Cap to close fitting.

Insert stabilizer leg with capped end here.

Step 4. After the frame has been constructed, verify you have built the proper side needed by standing it up next the step system.

Use the provided screws at the pre-drilled locations at the fittings.

Note: Turn the lower fittings shown above in photo 5 in the opposite directions for placement on other side of system.

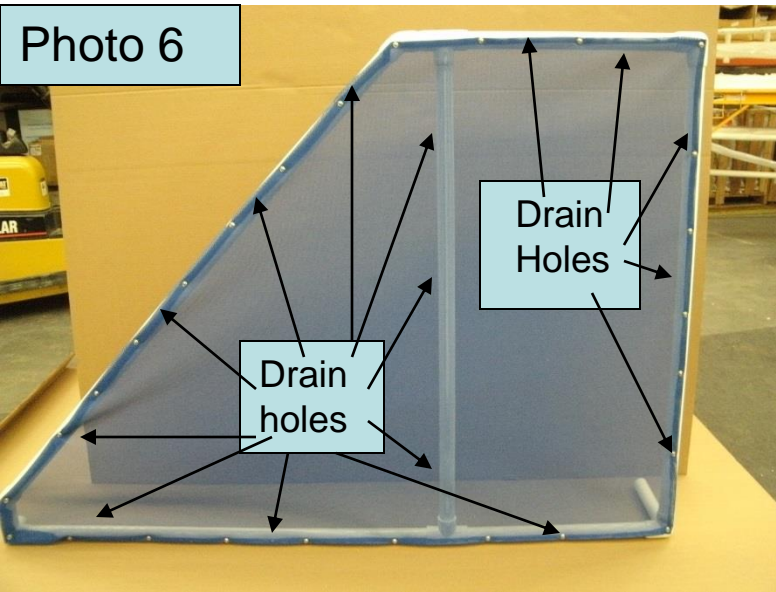

5. When the frame has been constructed, use a ¼” drill bit and drill drain-holes in the outer edge of the pipe at an angle for water to flow out. Drill holes about 12 inches apart.





6. Lay out the screen to match the frame. Starting at the upper Right corner of the frame and screen in Photo 9. First fold over the top side approximately ½” – Photo 7 Then fold the back side inward approximately ½” – Photo 8

7. Place edge-folded screen as shown in Photo 9 on fitting and use #8 screw with cupped washer as seen in Photo 10

Continue along the top of the screen folding over the top edge approximately ½” and use #8 screws with cupped washers approximately every 5 or 6 inches

BE SURE TO KEEP THE SCREEN TIGHT AS YOU GO.

*Note: your side screen may appear differently but the process is the same.

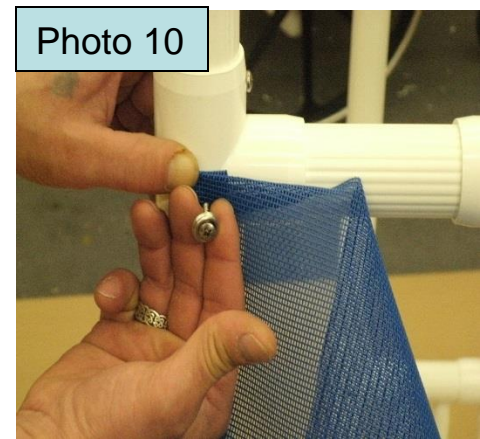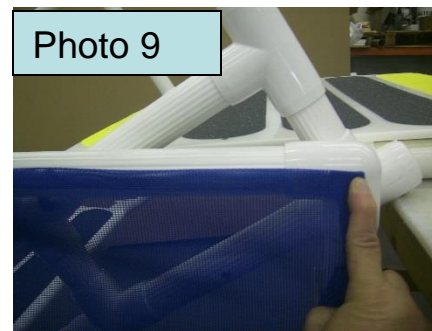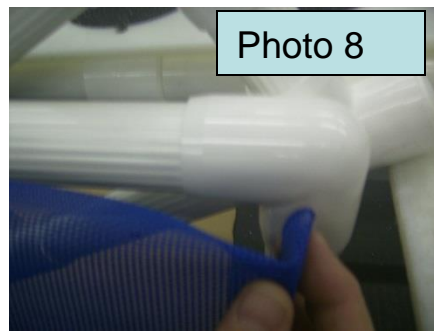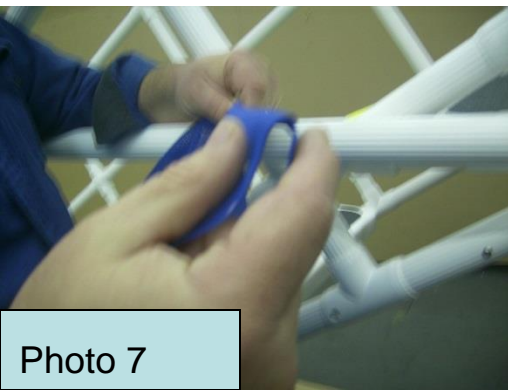

8. Locate the quick-clip assembly. Assemble but DO NOT screw yet.

9. Attach quick-clips to the lower and the upper end of the step system and side skirt. Somebody will need to get wet as the steps will need to be in the pool for this final step.



Screw Divot

10. Quick-clip locations shown are approximations, locate the best places for the clips to attach.

11. After the quick-clip locations are found AND the Positioning of the clips are determined; Use #8 screws in the divots of the clip and attach to skirt and steps.

DO NOT SCREW THE CLIPS TO THE STEP SYSTEM.

DO NOT use screws here

DO NOT use screws here





Comments and Suggestions for Safe and Convenient Use:

1. ***** Use a floating Barrier along the side of the unit to create a “No Crossing Zone” *****
2. **Place the ladder or step system next to the pool wall so that people cannot get between the ladder and the pool wall.**
3. Do not move the bottom of any access product from side to side while it is secure on the pool deck. This will put undue pressure on the platform. The result will be broken components and will render your product useless.
4. Be sure that the platform is resting uniformly on the pool deck. If it teeters or is in a bind after the unit is installed, the platform is not properly fit to pool deck. Use plastic shims and place under the edge of the platform in the appropriate place.
5. Products are designed for use by one person at a time. Do not exceed 550 pounds on any access product at one time.
6. Do not use any access product unless it is anchored to the deck.
7. Ladder legs may need to be trimmed for an accurate fit to pool height. Use a hacksaw or carpenter saw to trim the legs to desired length. Remember to add 1-1/4 inches to the measurements so that they will fit inside pipe fittings.
8. Keep your new AquaTrek Ladder clean. Use an industrial strength cleaner to clean your ladder every month. The solution known as CLR (Calcium, Lime, Rust) or Lime-A-Way are recommended cleaners. Keeping your new AquaTrek Ladder clean will preserve the plastic and add years to its life.

Additional Notes for Modified Units:

If you have a modified platform, you may need to custom cut any leg extensions to fit your pool. You might also have a U-bracket to clear specific deck configurations.

Product Warranty:

Rehab Systems, LLC warrants each Pool Access System and every part thereof to be free from any defect of workmanship or material for a period of one (1) year after delivery. During that one-year warranty time, Rehab Systems, LLC shall replace or repair any defective part free of charge to the purchaser, when defective part is returned to Rehab Systems, LLC. Transportation is prepaid by the purchaser. The warranty referred to above is the only warranty applicable to the equipment. It is expressly in lieu of any warranties or conditions otherwise implied by law including, but not limited to, implied warranties or merchantability or fitness for a particular purpose, the remedies under such warranty shall be the only remedies available to buyers or any other person. Manufacturer shall not be liable for loss of use of the equipment, loss of time, inconvenience or other consequential damage.

Terms and Conditions:

All products are shipped prepaid F.O.B. our shipping point. Prices subject to change without notice. Deliveries subject to availability. Back orders may become necessary from time to time. Payment requires ½ down with order, the other ½ within 30 days. Balance due prior to shipping. Some deck and gutter systems require special modifications and a charge may be added to the cost of your Ladder or Step system. Shipping & Handling: Rates may apply to shipments within the Continental U.S. only. Call our customer service line at (800) 726-8620 for rates outside the U.S.

Please Remit All Warranty Claims or Questions to:

CALL CUSTOMER SERVICE: 800-726-8620 or E-MAIL US: info@aquatrek2.com

MAIL US: VIP Solutions, LLC @ 3309 Fiechtner Drive Unit 3 Fargo ND 58103